

Renovar™

Advanced Skin and Wound Cleanser Advanced Wound Irrigation Solution

Instructions for Use

INGREDIENTS

Electrolyzed Water, Sodium Chloride (NaCl), and preservatives: Sodium Hypochlorite (NaOCl), and Hypochlorous Acid (HOCl).

NOTE: The solution contains sodium hypochlorite and hypochlorous acid as preservatives to reduce/prevent the growth of microorganisms within the solution.

DESCRIPTION

Renovar™ Advanced Skin and Wound Cleanser and Renovar™ Advanced Wound Irrigation Solution is a non-irritating, non-cytotoxic, non-sensitizing, non-oily, non-foaming pH neutral formulation.

INDICATIONS FOR USE

Renovar Advanced Skin and Wound Cleanser and Renovar Advanced Wound Irrigation Solution are intended for the cleansing, irrigation, moistening, debridement, and removal of foreign material and debris from exudating wounds, acute and chronic dermal lesions including stage I-IV pressure ulcers, stasis ulcers, diabetic ulcers, post-surgical wounds, first- and-second-degree burns, abrasions, minor irritations of the skin, diabetic foot ulcers, ingrown toenails, grafted/donor sites and exit sites. It is designed to moisten and lubricate wound dressings and devices used to irrigate wounds.

CONTRAINDICATIONS

None, however, there may be a temporary burning or stinging sensation when first applied to broken skin.

WARNINGS

- For external use only. Not for injection.
- For topical Use only.
- Federal Law restricts this device to sale by or on the order of a licensed healthcare practitioner.

PRECAUTIONS

- To be used only as directed by a licensed healthcare practitioner.
- Do not use to treat any condition other than that for which it is prescribed.
- Avoid contact with the eyes.
- If irritation occurs, discontinue use and consult a physician.

SIDE EFFECTS (POSSIBLE)

There may be a temporary burning or stinging sensation when first applied to broken skin.

DIRECTIONS FOR USE

To clean and aid debridement of wounds and peri-wound tissue. Do not rinse. To moisten a dressing, apply to wound dressing and cover wound bed. To keep dressing moist, apply Renovar as needed throughout the day.

HANDLING

No special handling precautions required.

STORAGE CONDITIONS

Store with cap closed tightly after each use in its original container. Store at 59°F to 77°F (15°C to 25°C) away from direct sunlight and heat sources. Do not freeze.

DISPOSAL

No special disposal considerations required.

KEEP OUT OF THE REACH OF CHILDREN.

Distributed by:
HR HealthCare
2600 Eastern Blvd., Suite 201
York, PA 17402 USA
hrhealthcare.com



HR HealthCare, the HR HealthCare logo, and Renovar are trademarks or registered trademarks of HR HealthCare.
©2022, 2024 HR HealthCare. All rights reserved.

LBL-0024 Rev. 1 (01/2024)