

Surgilube®

Sterile Surgical Lubricant

en

4. foilpac (REF # 281020543, 281020545 & 281020557)
 - a. Open foilpac at tear notch perforation.
 - b. Apply amount needed to complete procedure to provide adequate lubrication directly to the patient or instrument.
 - c. Discard package after use.

STORAGE CONDITIONS

Store at room temperature (15° - 30° C or 59° - 86° F).

Instructions for Use

REF

281020502	56.7 g (2 oz.) Screw Cap Metal Tube
281020512	56.7 g (2 oz.) Flip-Top Tube
281020536	120.49 g (4.25 oz.) Screw Cap Metal Tube
281020537	120.49 g (4.25 oz.) Flip-Top Tube
281020543	3 g foilpac® Packet
281020545	5 g foilpac® Packet
281020555	5 g Snap-Off Tip Metal Tube
281020557-288	31 g (1.1 oz.) foilpac® Packet

DESCRIPTION

Surgilube® sterile surgical lubricant may be used where a sterile, water soluble, non-staining lubricating jelly is indicated. Surgilube will not affect surgical instruments, rubber or plastics.

INDICATIONS FOR USE

Surgilube sterile surgical lubricant is intended to facilitate the entry of a diagnostic or therapeutic device into a body orifice by reducing friction between the device and the body orifice. Surgilube can be used in gynecological, urological, and general surgical procedures including pap tests, pelvic exams, colonoscopies, endotracheal tube insertions, enema insertions and lithotripsy.

CONTRAINDICATIONS

Sensitivity to ingredients.

WARNINGS


- Re-use of Single-Use products could result in cross-contamination, risk of infection, or adverse reaction for the patient.
- Consult your physician or institutional procedures on requirements for use of sterile products for the procedure being performed.
- Do not use in eyes and ears.
- Contains Chlorhexidine Gluconate. If irritation occurs, discontinue use, and consult a physician. **NOTICE:** Any serious incident that has occurred in relation to this device should be reported to HR HealthCare.

PRECAUTIONS

- Do not use if the product is opened, damaged, or beyond expiration date.
- Do not use if seal under cap is broken or missing (REF #'s 281020502, 281020512, 281020536, 281020537).
- Use of a lubricating jelly for a task other than that for which it is intended could result in risk to the patient or one which provides an unsatisfactory performance.

STEP-BY-STEP INSTRUCTIONS

1. **Laminate Flip-Top Tubes (REF # 281020512 & 281020537)**
 - a. Sanitize tube tip and cap prior to opening.
 - b. Remove cap, remove foil seal and replace cap prior to use.
 - c. Apply amount needed to complete procedure to provide adequate lubrication directly to the patient or instrument.
2. **Metal Tubes (REF # 281020502 & 281020536)**
 - a. Sanitize tube tip and self-piercing cap prior to opening.
 - b. Puncture metal seal with tube cap.
 - c. Apply amount needed to complete procedure to provide adequate lubrication directly to the patient or instrument.
3. **Metal Single-Use Tube (REF # 281020555)**
 - a. Sanitize tube tip prior to opening.
 - b. Use force to break the tube tip.
 - c. Apply amount needed to complete procedure to provide adequate lubrication directly to the patient or instrument.

 HR HealthCare
2600 Eastern Blvd., Suite 201
York, PA 17402 USA
877-302-1110
hrhealthcare.com



hrhealthcare.com/symbolsglossary

HR HealthCare, the HR HealthCare logo, Surgilube and foilpac are trademarks or registered trademarks of HR HealthCare.
©2015, 2024 HR HealthCare. All rights reserved.

LBL-0011-EN Rev. 3 (01/2024)

 HR HealthCare