

# HR<sup>®</sup> Lubricating Jelly

## HR<sup>®</sup> Lubricating Jelly 2X

en

### Instructions for Use

#### REF

201	113 g (4 oz.) Flip-Top Tube, Sterile
203	56.7 g (2 oz.) Flip-Top Tube, Sterile
205	36 g (1.25 oz.) OneShot <sup>®</sup> Packet, Sterile
205ST	36 g (1.25 oz.) OneShot <sup>®</sup> Packet with SafeWrap <sup>®</sup> Package, Sterile
206	10 ml Pre-filled Syringe, Sterile
207	3 g OneShot <sup>®</sup> Packet, Sterile
208	3 g OneShot <sup>®</sup> Packets in CarePac <sup>®</sup> Box, Sterile
209	5 g OneShot <sup>®</sup> Packet, Sterile
210	56.7g (2 oz.) 2X, Flip-Top Tube, Sterile
211	36 g (1.25 oz.) 2X, OneShot <sup>®</sup> Packet, Sterile

#### DESCRIPTION

HR<sup>®</sup> Lubricating Jelly is a sterile lubricating gel that is water-soluble, stable, non-greasy, and spreads evenly. It is widely used in hospitals and physicians' offices and is ideal for lubricating rubber products, rectal thermometers, and rectal and vaginal nozzles for infants and adults.

#### INDICATIONS FOR USE

HR Lubricating Jelly is a sterile gel intended to facilitate the entry of a diagnostic or therapeutic device into a body orifice by reducing friction between the device and body; it is not dedicated to a particular type of anatomy.

#### CONTRAINDICATIONS

Sensitivity to ingredients.

#### WARNINGS AND PRECAUTIONS

- Do not use product beyond labeled expiration date as sterility and performance cannot be guaranteed.
- Re-use of the HR Lubricating Jelly Product (REF #'s 205, 205ST, 207, 208, 209, 211) could result in cross-contamination, risks of infection, or adverse reaction for the patient.
- A very small percentage of men and women may be sensitive to one or more of the ingredients in this lubricant.
- Keep out of reach of children.
- If irritation occurs, discontinue use and consult a physician.
- Do not use product if package is damaged or opened.
- Do not use if seal under cap is broken or missing (REF#'s 201, 203, 210).
- Use of a lubricating jelly for a task other than that for which it is intended could result in risk to the patient or one which provides an unsatisfactory performance.

#### PRE-USE, HANDLING, AND INSPECTION

The product must be thoroughly inspected upon receipt and prior to use to assure packaging is not damaged or defective. If any product damage or leakage is identified, the product should be discarded.

#### STEP-BY-STEP INSTRUCTIONS


1. **HR Lubricating Jelly OneShot SafeWrap (REF # 205ST)**
  - a) Sanitize exterior of SafeWrap pouch prior to introduction to sterile environment.
  - b) Remove the SafeWrap outer pouch and place the HR Lubricating Jelly packet on the sterile tray.
  - c) Apply an amount of HR Lubricating Jelly directly to the patient or instrument as needed.
  - d) Discard package after use.
2. **HR Lubricating Jelly OneShot Packet (REF #'s 205, 207, 208, 209, 211) & HR Lubricating Jelly Syringe (REF # 206)**
  - a) Apply an amount of HR Lubricating Jelly directly to the patient or instrument as needed.
  - b) Discard package after use.

#### 3. HR Lubricating Jelly Flip-Top Tube (REF #'s 201, 203, 210)

- a) Sanitize cap and tip prior to opening. Remove tamper-evident seal.
- b) Flip cap to open.
- c) Apply an amount of HR Lubricating Jelly directly to the patient or instrument as needed.

#### RECOMMENDED STORAGE CONDITIONS

- Store in a clean, dry place at room temperature (15°-30°C or 59°-86°F).

 HR HealthCare  
2600 Eastern Blvd., Suite 201  
York, PA 17402 USA  
hrhealthcare.com



HR HealthCare, the HR HealthCare logo, HR, OneShot, SafeWrap, and CarePac are trademarks or registered trademarks of HR HealthCare.  
©2008-2025 HR HealthCare. All rights reserved.

LBL-0017-EN Rev. 3 (02/2025)